



## Quick Reference Guide

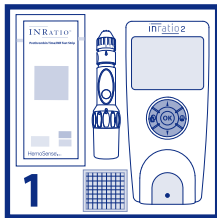


1

Make sure Test Strip is at room temperature and prepare other supplies.

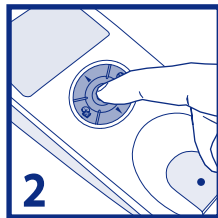


2

Turn on the monitor, insert Test Strip and check the Strip Code.

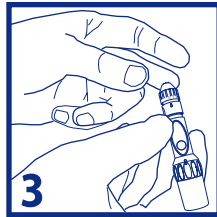

3

Perform a fingerstick when the monitor prompts you to apply a drop of blood.

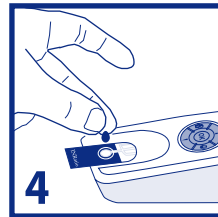

4

Apply a large, hanging drop of blood to the sample well of the Test Strip.



5

The results will appear when the test is finished.

## Step by Step Procedure

1. Make sure Test Strip is at room temperature.
2. Prepare other supplies:
  - Monitor
  - Alcohol Pads
  - Gauze Pads
  - Bandages
  - Lancets
  - Results Book
  - Puncture Proof Container
3. Warm hands to increase circulation.
4. Turn on monitor by pressing the OK button.
5. Insert Test Strip at prompt, check the Strip Code. If needed, change the Strip Code by pressing the UP/DOWN button. Press OK to change a digit. Press OK again to proceed with the test. Wait for the monitor to warm up.
6. Perform a fingerstick when the monitor prompts you to apply a drop of blood.
7. Apply a large hanging drop of blood to the Test Strip, over the green light.
8. Wait for result to appear on the display.
9. Record results.
10. Discard all materials that have come in contact with blood in a puncture proof container.

HemoSense...



Biosite Incorporated  
9975 Summers Ridge Road  
San Diego, CA 92121 USA  
+1-877-441-7440 (USA toll free)  
+1-321-441-7200 (tel)

INRatio and HemoSense are registered trademarks of the Inverness Medical group of companies.  
© 2009 Inverness Medical. All rights reserved.

0200522 REV B